



dry-cap uv 2.2[®]

Electronic drying cap for all rechargeable hearing aids.

INSTRUCTION FOR USE

| | | |
|----|-----------------------------|---------|
| EN | INSTRUCTION FOR USE | 2 - 3 |
| DE | GEBRAUCHSANWEISUNG | 4 - 5 |
| FR | MODE D'EMPLOI | 6 - 7 |
| ES | MANUAL DE INSTRUCCIONES | 8 - 9 |
| PT | MANUAL DE INSTRUÇÕES | 10 - 11 |
| IT | MANUALE DI ISTRUZIONI | 12 - 13 |
| NL | GEbruIKSAANWIJZING | 14 - 15 |
| GR | ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ | 16 - 17 |
| TR | KULLANIM KILAVUZU | 18 - 19 |
| DK | BRUGERVEJLEDNING | 20 - 21 |
| SE | BRUKSANVISNING | 22 - 23 |
| HU | HASZNÁLATI ÚTMUTATÓ | 24 - 25 |
| PL | INSTRUKCJA OBSŁUGI | 26 - 27 |
| SL | NAVODILA ZA UPORABO | 28 - 29 |
| HR | UPUTE ZA UPOTREBU | 30 - 31 |
| RU | РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ | 32 - 33 |
| CN | 用法說明 | 34 - 35 |
| JP | 取扱説明書 | 36 - 37 |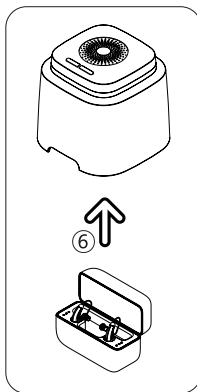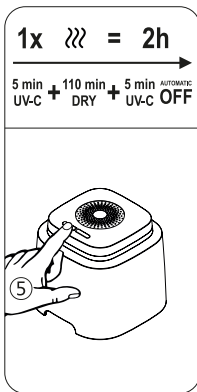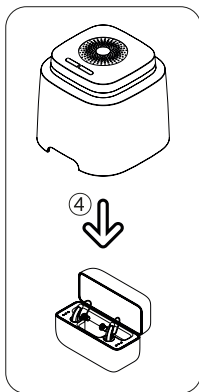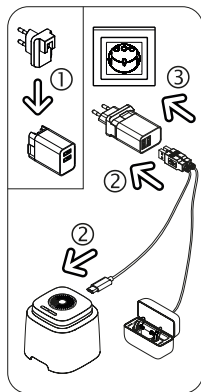

EN INSTRUCTION FOR USE

???

Dry&Clean Mode (2 h)

The *Dry&Clean* mode is an automatic program and combines fast drying with an efficient fan with UV-C light at the beginning and end of the drying time for perfect hygiene.

☀

Clean Mode (5 min)

The *Clean* Mode is a short-term hygiene program and starts the optimized UV-C light, which effectively eliminates up to 99.99% of germs, bacteria and fungi present in the household.

Using first time

1. Insert your country-specific adapter into the slot of the supplied USB universal power supply (5V/2.5A).
2. Plug the USB-A side of the USB cable into the USB-A socket of the power supply and the USB-C side of the cable into the USB-C socket on the back of your dry-cap uv 2.2®.
Optional: The second USB-A port on the power supply can be used for the hearing aid charging station.
3. Then plug the USB power supply into a power outlet. ① The dry-cap uv 2.2® is ready to use

Operation ??? **Dry&Clean Mode (2 h)**

4. Place the dry-cap uv 2.2® with the acrylic hood over the charging station with the rechargeable hearing aids.
5. Press the left button (*Dry&Clean* mode) and the care program (warm drying + UV-C) starts automatically.
 - The drying by fan with warm air (up to 35°C / 95°F) is controlled by a microprocessor and runs for 2 h
 - Ultraviolet light (275 nm) eliminates germs and bacteria to 99.99% within 5 min ① LED blink light
 - After 5 min the UV-C light switch off automatically, the drying process continues ① LED permanent light
 - Within the last 5 minutes of the 2 h drying program, a second UV-C hygiene runs ① LED blink light
 - After 2 h the dry-cap uv 2.2® automatically shutdown to stand-by ① LED switch off

Drying and hygiene are complete, the charging of the hearing aids in the charging station continues and can remain under the dry-cap uv 2.2® after the charging station is switched off.

6. Lift the dry-cap uv 2.2® up and off next to the charging station. Remove the dried and hygienically cleansed hearing aids from your charging station.

Operation Clean Mode (5 min)

- Put the dry-cap uv 2.2[®] with the acrylic hood over the products to be cleansed.
- Press the right button (*Clean Mode*) and the hygiene program (UV-C) starts automatically.
 - Ultraviolet light (275 nm) eliminates germs and bacteria to 99.99% within 5 min ① LED blink light
 - After 5 min the dry-cap uv 2.2[®] automatically shutdown to stand-by ① LED switch offHygiene is complete.
- Lift the dry-cap uv 2.2[®] and place it next to the products. Remove the hygienically cleansed products.

Maintenance and operation

We recommend wet wipes for cleaning your dry-cap uv 2.2[®]. Please also read the care and operating instructions for your hearing aids and charging station. The dry-cap uv 2.2[®] is very energy efficient; unplug the power supply only for prolonged absence. You can interrupt the drying process in *Dry&Clean* mode by holding the button for more than 3 seconds or by lifting and tilting the unit.

Warnings

Avoid contact with water. Do not use the dry-cap uv 2.2[®] near sinks, etc. In the event of a malfunction, unplug the power cord and take the dry-cap uv 2.2[®] to your dealer. Do not disassemble the dry-cap uv 2.2[®] and only use the USB power supply unit supplied. Children must be supervised to ensure that they do not use the dry-cap uv 2.2[®] as a toy. The dry-cap uv 2.2[®] is not intended for use by persons with limited physical, sensory, or mental abilities or lack of experience and/or knowledge, unless they are supervised by a person responsible for their safety or have been instructed by that person on how to use the dry-cap uv 2.2[®]. CAUTION: Do not look into the UV-C light, as this may damage your eyes and skin. The dry-cap uv 2.2[®] is equipped with a 3G motion sensor. If the dry-cap uv 2.2[®] is moved, turned, or tilted during use, the 3G motion sensor will switch off the device! In idle mode, you must then restart the device. Please only use with UV-C cleanable products, never with humans or animals!

Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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© 2010-2026 flow-med GmbH. All rights reserved.
flow-med GmbH
Otoplastic AG